



Spilfix

MAKING A CLEANER PLANET



BLOOD SPILL KIT 5 Ltr

SPLSK05BLD

ABSORBENCY : 5 Ltr

CONTAINS :

- › One Set of Personal Protective Equipments
 - › Safety Goggles : 1 Nos
 - › 3Ply MB Mask : 1 Nos
 - › Cotton Cargo Apron : 1 Nos
 - › Safety Gloves : 1 pair
 - › Disposable Shoe Cover : 1 pair

- › Blotting Paper : 5 Pcs
- › Disinfectant Liquids : 500 ML
- › Scoop & Brush : 1 Nos
- › Baricadding tape : 1 Roll
- › Clinical Wastage Bag : 5 Nos
- › Blood Neutralizer : 1000 Gms
- › Caution Mat Yellow : 2 nos
- › Chemical Absorbent Pads (Green & Blue): 2 Nos
- › Rejected X-Ray Flim : 2 Nos

SPIFIX SPLSK05BLD Blood Spill Kits Specially Customized design and packed for quick & Efficient Response against large Large Spillage.

- › Put on Protective Gears Supplied with Kit.
- › Sprinkle Granular over the Spill and leave for 2 min.
- › Once Granules have absorbed the spill, collect same using x-ray sheet & Scoop and brush.
- › Sanitise the Spill area with disinfectant liquids.
- › Clean the Spill area using Mats & Pads.
- › Bag and contents must be disposed of in accordance with local waste disposal policies for biohazard waste.
- › As per NABH standard and satisfied ISO guidelines